

BLOODBORNE PATHOGEN

BODY FLUID CLEANUP KIT



PART #: FLUID

CONTENTS:

- ISOLATION GOWN
- FACE MASK
w/ eye shield
- NITRILE GLOVES
(1pair)
- RED-Z FLUID SOLIFIER
- SCOOP & SCRAPER
- DRY WYPES (2)
- SANITIZER TOWELETTE (2)
- BIOHAZARD BAG
24" x 24"
- SHOE COVERS
(1pair)
- GERMICIDAL TOWELETTE
- SPECIMEN BAG
- INSTRUCTIONS



Clean Plus, Inc. / CPI Divisions
138 East Main Street PO Box 678
West Concord, MN 55985
www.cpidivisions.com



Always, the Pride of Quality
Inspect for Replacement 6/2015

BODY FLUID CLEANUP KIT

This kit is designed for use on potentially infectious spilled body fluids such as blood, urine, vomit and feces. The following simple steps are recommended for fast, safe clean-up:

INSTRUCTIONS

1. Apply MEDICAL GLOVES AND APPROPRIATE PROTECTIVE CLOTHING
2. Sprinkle RED-Z over spilled area. Liquid will quickly congeal for safer handling.
3. Remove congealed material with SCOOP and SCRAPER. Carefully place in SPECIMEN BAG. Place specimen bag in RED BIOHAZARD BAG. Clean away remaining solids with DRY WYPE.
4. Disinfect affected surface area with Sani-Cloth™ and DRY WYPE
5. Place all contaminated materials (including gloves) in RED BIOHAZARD BAG. Seal and dispose of bag as required by local regulations for infected solid waste. Wipe hand(s) with ANTIMICROBIAL HAND WIPE. Wash with soap and running water as soon as possible.